

Instructions for the Installation of New Batteries and Battery Replacement Label for the ZOLL AED Plus

Please Note: Replace the batteries in your AED Plus three years from the installation date of new batteries.

1. Ensure that the AED Plus is turned off. Open the battery compartment by removing the battery cover from the back of the unit.
2. Remove all batteries at once and dispose of old batteries properly. Place new batteries into the battery bank, observing battery polarity markings and making sure that all batteries are securely seated and properly oriented. After placing the first 5 to 9 batteries in the battery well, the INSTALL BATTERIES audio prompt reminds you to install the remaining batteries in the battery compartment.
3. After installing new batteries, press the Battery Reset Button inside the battery compartment when prompted. Pressing the button resets the battery usage indicator to full charge.
4. Close the battery compartment.
5. Turn the AED Plus over.
6. Locate battery replacement label which is included in the battery kit.
7. Write the battery replacement date on label. (Three calendar years from today's date.) Example: If today's date is (03/22/2009) the replacement date will be (03/22/2012).
8. Peel label from paper.
9. Affix label onto AED Plus below the "Status Indicator", as displayed below.

Explanation of Symbols:

 - Replace batteries on or before <date>




Place the AED Battery Replacement Label
below the AED Plus Status Indicator

